



Lento PST® System

Instructions for Use - Uploading MRI to Lento Medical Database (MRI Center)

Software Release 1.3.x

Document ID: LB-72-01-002

Model: Revision B

September 5, 2019

Author:

Signature:



Date

9/5/2019

Reviewed and Approved by:

Signature:



Date

9/5/2019

This is the reference guide for the MRI Technologist as an aid in uploading the MRI scanning images to the Lento Medical cloud database. There is a separate full MRI reference guide for use in positioning and patient orientation. Also, a quick MRI setting reference guide is available for use by experienced technologists and both are available upon request if they have not already been supplied or is misplaced.



WARNING: It must be noted that the use of MRI scanners in patients with metallic implants in or near the knee joint may adversely affect the quality and accuracy of the images obtained. For this reason it is generally recommended that such MRI scans not be attempted.



WARNING: The use of MRI scans for making custom cutting guides in pediatric patients has not been studied and the results of the use of Lento Medical cutting guides in these patients is unknown. For this reason it is generally recommended that such MRI scans not be attempted. Please verify the MRI request with the ordering physician.



Manufacturer:

Lento Medical, Inc.
15110 Northwest Freeway, Suite 150
Houston, TX 77040 USA
+1 (510) 413-3230



The following organization is the Authorized Representative for the Lento Medical PST® System:



HealthLink Europe BV
De Tweeling 20-22
5215 MC's - Herrogenbosch
The Netherlands
Tel.: +31-13 5479300
Web: www.healthlinkeurope.com


Introduction

These instructions are intended to guide the MRI Center to upload MRI files through the Lento Medical web portal (www.lentomedical.com/login).

1. Log In

Lento Medical Admin or Sales personnel will assist the MRI representative with registering an account for the Lento Medical web portal. Once a username and password is created, the surgeon will use the information to log in.

- a. Enter Username
- b. Enter Password
- c. Click "Login" button

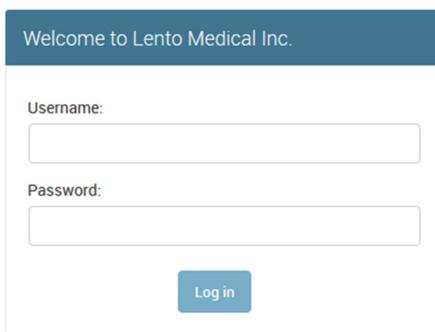

Welcome to Lento Medical Inc.

Username:

Password:

Log in

2. MRI Center Landing Page

The MRI representative landing page will appear listing all active cases for MRI representative view. The status of a case that requires MRI upload will show "No MRI Files." Surgeon can double click the case number and the MRI upload page will appear.

MRI Center Cases

1234 ABC AVE, Fremont, CA 95000

Case #	Surgeon	Patient	DOPO	DOS	DOB	Type	Side	Notes	Status
K27915R	Jane Doe	John Smith	Aug. 11, 2016	Oct. 15, 2016	June 16, 1953	TKA	R		No MRI Files


3. View Surgeon Planning Reference

From the MRI upload page, MRI representative will need to click "Choose File." Standard browser will pop up to select MRI folder or select multiple MRI files to be upload it. Once selection is done, click "OK." Once files are selected, click "Submit" button to upload it Lento Medical database.

Upload MRI Data for K27915R John Smith

Select Directory with MRI Files

No file chosen

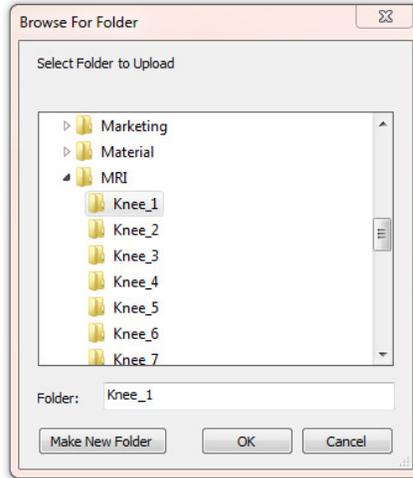

4. MRI upload confirmation

Once MRI upload completed without any errors, below screen will show.

K27915R John Smith

Surgeon Jane Doe Email Address akim.orthomade@gmail.com Phone 1234567890 Fax N/A Office Name OM Office Office Address 402 Kato Terrace, Fremont, CA 94539	Patient John Smith Age 63 Gender Male DOPO Aug. 11, 2016 DOB June 16, 1953 DOS Oct. 15, 2016 Surgery TKA Side Right	MRI Center OrthoMade MRI Center 1234 ABC AVE, Fremont, CA 95000 Phone 5105551234 Fax 5105551234
--	---	--

Note
No notes.

Case Files
MRI Data
K27915R_mri
46.6 MB

If there is any error, system will prompt to reload the MRI Files.